

Flexi-Q eMU-C

Electromechanical Multi-use Auto-injector for drugs in cartridges





Smart features, wireless connectivity

Technical Data

Device description	Electromechanical multi-use au- to-injector for drugs in cartridges
Weight (excl PFS/drug)	125g
Grip area	(width) 42mm - (thickness) 29mm
Primary container	Compatible with standard 3 ml / 1.5 ml cartridge Dual chamber cartridge
Route of administration	Subcutaneous
Drug formulation	Liquid, lyophilised or powdered
Power source	Rechargeable battery



Handling Instructions

- 1  Insert Cassette into Device port
- 2  Remove Cap Attach needle
- 3  Push Device against injection site and press Inject button
- 4  Prime (once for each new cartridge)
- 5  Eject Cassette Detach needle and dispose of in appropriate container.

Key Features



Fully automatic
injection process



Advanced
user interface



Smart device
with wireless
connectivity

Medical & Dosing

Dosing options	Fixed or user-selectable
Injection duration	3 sec – 15 sec (fixed or user-selectable)
Viscosity	Up to 60cP
Injection volume range	0.1mL – 1.0mL
Injection volume increment	0.05mL
Maximum injection volume	1.0mL

Needle

Needle shield	Rigid needle shield (RNS) or soft needle shield (SNS)
Activation method	Injection button
User guidance	On-screen step-by-step instructions (text and icon)
User feedback	Visual (on-screen) and audible indicators
Number of user handling steps	4
Device customization options	External geometry and colors